# Qwiek<sup>®</sup>.embrace

Quick start & safety instructions

-  Snelstart & veiligheidsinformatie
-  Schnellstart & Sicherheitsinformationen
-  Hurtig start & sikkerhedsoplysninger
-  Démarrage rapide & informations de sécurité
-  Avvio rapido & informazioni sulla sicurezza

Qwiek<sup>®</sup>

# Qwiek<sup>®</sup>.embrace

A comforting embrace for moments  
of stress and anxiety

<b>EN</b> The Qwiek.embrace	<b>4</b>
Intended use .....	4
Contraindications .....	4
Package contents .....	4
Product description .....	5
<b>Cleaning &amp; maintenance</b>	<b>5</b>
<b>Getting started with the Qwiek.embrace</b>	<b>6</b>
<b>Symbols on the product</b>	<b>8</b>
<b>Safety, warnings, and precautions</b>	<b>8</b>
<b>Technical specifications</b>	<b>9</b>
<b>Warranty &amp; service</b>	<b>9</b>
<b>Environmental information &amp; disposal</b>	<b>9</b>
<b>Cybersecurity &amp; data privacy</b>	<b>9</b>
<b>Troubleshooting</b>	<b>10</b>

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## A comforting embrace for moments of stress and anxiety

### **Congratulations on purchasing the Qwiek.embrace.**

This device has been carefully developed in collaboration with healthcare professionals to provide peace, comfort, and relaxation. The Qwiek.embrace is a medical device that combines three sensory stimuli: deep pressure stimulation, rhythmic vibrations, and meditative sounds.

Please read this user manual carefully before using the product. It contains important information for safe and proper use, contributing to your safety and that of others. Keep this manual for future reference. Use this device only as directed by a qualified healthcare professional or therapist.

If you need assistance with setting up, using, or maintaining the Qwiek.embrace, or if you encounter unexpected situations, please contact the supplier.

Report any serious incidents involving the Qwiek.embrace immediately to the supplier and the competent authority in your country.

### **Intended use**

The Qwiek.embrace combines deep pressure stimulation from a weighted cushion with perceivable vibro-acoustic vibrations and sound. The device provides sensory stimuli that can contribute to relaxation and a sense of security, supporting overall well-being and promoting calm during moments of stress or agitation.

The device is suitable for individuals who benefit from this type of sensory stimulation. It is designed for use in care environments (such as nursing homes and hospitals) as well as at home. Thanks to its simple operation, the Qwiek.

embrace is suitable for independent use as well as a tool for healthcare professionals. Use should always be by or under the supervision of qualified caregivers familiar with the operation of the product and this manual.

This device is not intended to diagnose or treat any medical condition or disease.

### **Contraindications**

Do not use this device on individuals with active electronic implants, such as a pacemaker or implantable cardioverter-defibrillator (ICD).

Refer to the safety instructions later in this manual for additional important information and situations where consultation with a physician is necessary.


### **Package contents**

Check that the packaging is intact and that all parts are present. Do not use the device if you notice any damage; in that case, contact the supplier.

- 1x Qwiek.embrace weighted cushion
- 1x USB charging cable
- 1x USB power adapter
- 1x user manual

### Product description

1. **Weighted cushion:** provides even deep pressure stimulation.
2. **Control panel:** contains the buttons and LED indicator for easy operation.
3. **Tactile audio Unit:** the internal system that generates both the vibrations and the sound.
4. **USB-C charging port:** used to charge the internal battery with the supplied charging cable and adapter.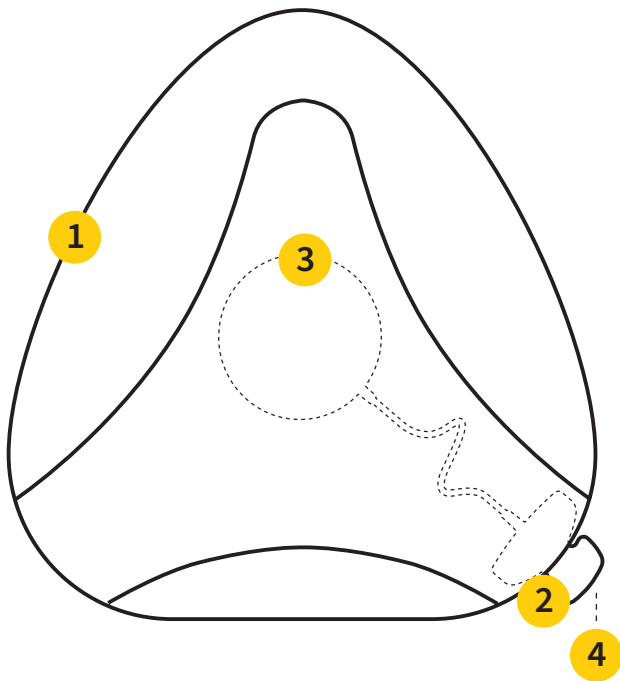


### Cleaning & maintenance

- ⚠️ **WARNING:** always disconnect the device from the charger before cleaning. Never submerge the inner cushion containing the electronics in water.
- Always clean the Qwiek.embrace before using it with another person.
- The removable outer cover can be washed. Follow the instructions on the care label inside the cover.
- The inner cushion can be wiped with a slightly damp cloth. For disinfection, you may use products such as alcohol (70%) or a mild chlorine solution (250 ppm). Always follow the instructions provided with the disinfectant.
- The charging cable and power adapter can be cleaned with a soft, dry, or slightly damp cloth.
- ⚠️ **CAUTION:** make sure all parts are completely dry before reassembling or charging the device to prevent short circuits.
- Regularly inspect the device, cover, and cables for signs of wear or damage. Stop using the device immediately if you notice any defects and contact the supplier.
- To maximize battery lifespan, recharge the device when the LED indicator remains solid red. If you do not use the device for an extended period, ensure the battery is about half full and recharge it at least once every 6 months to prevent deep discharge.

# Getting started with the Qwiek.embrace

Quick and easy in three steps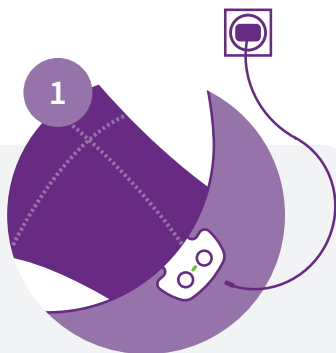

## Turning on the Qwiek.embrace


To activate the Qwiek.embrace for the first time, connect it using the included charging cable. Once powered on, you may unplug it. From then on, the Qwiek.embrace will automatically turn on whenever you pick it up.

## Playing music & vibrations

Press the Play/Pause button to start or pause the music on the Qwiek.embrace.

▶**2x** Press the button twice quickly to skip to the next playlist.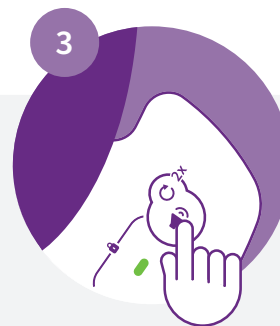

## Adjusting the volume


Use the volume button to cycle through three levels: **low**, **medium**, and **high**.

↻**2x** Press the button twice quickly to loop the current playlist.

- **Green pulsing:** the Qwiek.embrace is waiting for input
- **Green solid:** the Qwiek.embrace is playing
- **Orange:** the Qwiek.embrace has 50% battery remaining
- **Red:** the Qwiek.embrace is empty, please connect it to the charger
- **Purple blinking:** the Qwiek.embrace is locked

## Take it a step further for optimal use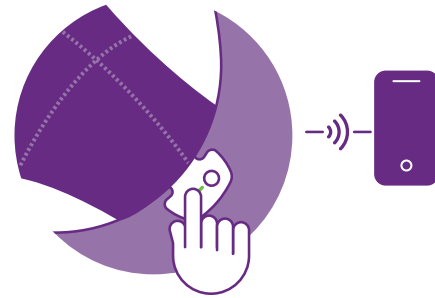

### Qwiek app (optional)

Control the Qwiek.embrace remotely and connect it to Wi-Fi. This ensures you always have the latest software and features.

Download the app: search for “Qwiek” in the Apple App Store (for iOS) or the Google Play Store (for Android). Make sure Bluetooth is enabled on your smartphone or tablet.

### Qwiek.embrace giving problems?

Scan the QR code or visit the support page: [help.qwiek.eu/qwiek-embrace](https://help.qwiek.eu/qwiek-embrace).

There you will find, among other things, the detailed user manual, instructional videos, and technical support, everything you need at your fingertips!

**Tip!** Charge the Qwiek.embrace overnight so you can fully use it again the next day!

Do you want  
the Qwiek.embrace  
to speak **your**  
**language?** Choose  
your language via  
the app.

The background image shows a woman with short grey hair and a patterned top leaning over an elderly woman with white hair and glasses. The elderly woman is holding a grey, textured, wedge-shaped device labeled 'Qwiek' on her lap. The device has a purple zipper-like detail along its edges. A purple circular callout box is overlaid on the image, containing text and two speech bubble icons with the flags of the United Kingdom and Hungary.

Symbols on the product	
	CE-marking: this product complies with the requirements of the Medical Device Regulation (EU) 2017/745 and the Radio Equipment Directive (RED) 2014/53/EU.  Qwiek BV hereby declares that the radio equipment type Qwiek.embrace is in compliance with Directive 2014/53/EU. The full text of the EU Declaration of Conformity can be found at the following website: <a href="https://qwiek.eu/compliance">https://qwiek.eu/compliance</a> .
	This symbol indicates that the product must not be disposed of with regular household waste. Please take the product to an authorized collection point for the recycling of electrical and electronic equipment to prevent harm to the environment and human health.
	Refer to the user manual for important safety information.
	Direct current
	Applied part, type BF
	Manufacturer and year of manufacture
	Serial number
	The device is protected against solid objects $\geq 12.5$ mm and against dripping water falling at an angle.
	<b>WARNING</b> Indicates situations that may cause personal injury or pose a health risk.
	<b>CAUTION</b> Indicates that certain actions may cause damage to the device or its accessories.

## Safety, warnings, and precautions

Please read all warnings and precautions in this manual carefully. They are intended to ensure safety, prevent injury, and avoid situations that could cause damage to the device.

### WARNING

- Consult your physician or therapist before using this device if the user is under medical treatment or if you are unsure whether this product is suitable for the situation.
- Always use the device under adult or professional supervision when used with children or individuals with cognitive impairments. Keep the device out of reach of unsupervised children.
- Do not use the device on damaged skin, such as open wounds, rashes, inflammations, or in cases of (risk of) pressure sores.
- Stop use immediately if the user feels discomfort, dizziness, or nausea.
- Do not use the device while it is charging. Always disconnect the charger before use.
- Inspect the device and cables for physical damage before each use. Stop use immediately if any defect is detected.
- Prevent contact with water or other liquids. Do not use the device in damp environments and do not touch the plug with wet hands.
- Lay the charging cable safely to prevent tripping hazards or risk of strangulation.
- Do not place open flames, such as lit candles, on or near the product. Keep the battery away from fire.
- If the product becomes abnormally hot, emits an unusual smell, or takes longer than usual to charge, stop using and charging it immediately and contact the supplier.
- Do not open, modify, or puncture the device or battery. Avoid skin or eye contact if the battery leaks. In case of contact, rinse immediately with plenty of water and seek medical advice.

#### **CAUTION**

- Use the device only for its intended purpose and only with original accessories supplied by Qwiek.
- Do not drop the device and protect it from shocks, extreme temperatures, moisture, and direct sunlight.
- Use only the supplied USB cable and power adapter, and ensure the mains voltage matches the specifications. If the adapter becomes defective or does not function properly, contact the supplier immediately for replacement or further instructions. Under no circumstances should another adapter be used without the supplier's explicit approval, in order to ensure the safety of both the user and the device.
- Do not charge the device under a blanket or in a poorly ventilated area to prevent overheating.
- Do not attempt to repair the device yourself. This voids the warranty. Contact the supplier for service.

## Technical specifications

#### **Qwiek.embrace:**

Model: QEM01

Dimensions: 550 x 530 x 200 mm

Weight: 2.5 kg

Battery type: 6600mAh Li-ion (3.6V, non-replaceable)

Battery lifespan: 2 years under normal use

Rated voltage: 5V DC

Rated power: 6 Watts

Protection rating: IP22, applied part type BF

Operating conditions: 5°C ~ 40°C, relative humidity 20% ~ 80%, atmospheric pressure 700 hPa ~ 1060 hPa

Storage conditions: 0°C ~ 50°C, relative humidity 10% ~ 90%, atmospheric pressure 700 hPa ~ 1060 hPa

Connectivity: 2.4GHz Bluetooth & Wi-Fi

Frequency band: 2402 - 2480 MHz

Maximum output power (EIRP): 20mW (13 dBm)

#### **Qwiek.embrace charger:**

Input: 100 - 240V, 50/60Hz

Output: 5V DC 2400mA

## Warranty & service

Qwiek B.V. warrants this product to be free from material and manufacturing defects for a period of two (2) years from the original date of purchase. This warranty applies only to the original purchaser and is non-transferable. The warranty does not cover damage caused by commercial, unreasonable, or improper use, normal wear and tear, or damage resulting from unauthorized

repair attempts. For questions, service, or support, please contact the supplier. Do not attempt to repair the device yourself.

## Environmental information & disposal

At the end of its life, this product must not be disposed of as regular household waste. The crossed-out wheellie bin symbol (WEEE) indicates that the product must be collected separately as electrical and electronic waste.

This product contains a built-in, rechargeable lithium-ion battery that cannot be removed by the user. Do not attempt to remove the battery yourself.

To protect the environment and conserve valuable resources, please take the entire product to a designated collection point in your municipality. You may also return the product to the supplier when purchasing a new, similar product. Proper disposal ensures that the device and battery are processed in an environmentally friendly manner, helping to prevent harm to people and the environment.

## Cybersecurity & data privacy

Since you can connect the Qwiek.embrace to the app, it's important to consider digital security. We take your privacy and the security of the product very seriously. With a few simple steps, you can help keep everything safe and secure.

The most important step is securing your smartphone or tablet. By using a passcode, fingerprint (Touch ID), or facial recognition (Face ID), you ensure that only you have access to the app.

Always download the Qwiek app from the official Apple App Store or Google Play Store. This ensures you have the correct, safe version.

We also recommend installing updates for the app and your phone as soon as they become available. These updates not only add new features but also contain important security improvements.

Updates for the Qwiek.embrace itself are delivered via the app; simply follow the on-screen instructions.

When connecting the Qwiek.embrace to Wi-Fi, use a trusted and secure network. We advise against using public, unsecured Wi-Fi networks.

Your privacy is of utmost importance to us. The Qwiek.embrace does not collect personal data. The usage data we collect is completely anonymous and linked only to the device's serial number. This helps us improve our service without knowing who you are.

## Troubleshooting

Problem	Possible cause	Solution
The Qwiek.embrace does not turn on when pressing a button or moving it.	The battery is empty.	Connect the product to a suitable USB-C charger to recharge.
The Qwiek.embrace does not turn on after connecting to a suitable charger.	Power supply issue.	Check the power supply. Try a different outlet or use another USB-C charger and cable.
The Qwiek.embrace is connected to power but does not turn on.	Internal error.	Use a pen or paperclip to press the reset button located inside. To access it, open both the outer and inner covers using the zipper.
No sound/vibration.	No files found.	Check the LED indicator; it will flash red.
Qwiek.embrace cannot be operated, LED flashes purple.	Button lock is enabled.	Press and hold both buttons to unlock the controls.

To reset the Qwiek.embrace to factory settings, press and hold both buttons for 10 seconds while connecting the charger. Release the buttons as soon as the light turns off.

For a more detailed troubleshooting guide, please refer to the online manual at [help.qwiek.eu](https://help.qwiek.eu) or scan the QR code.

# Three sensory stimuli that bring body and mind into balance

The Qwiek.embrace combines three sensory stimuli that reinforce each other. Together, they help the body to relax and the mind to find calm.


## Meditative sounds

Breathing, heartbeat, or nature sounds: tones that calm the nervous system.

## Deep pressure stimulation

Provides the body with a tangible boundary and a safe, comforting feeling.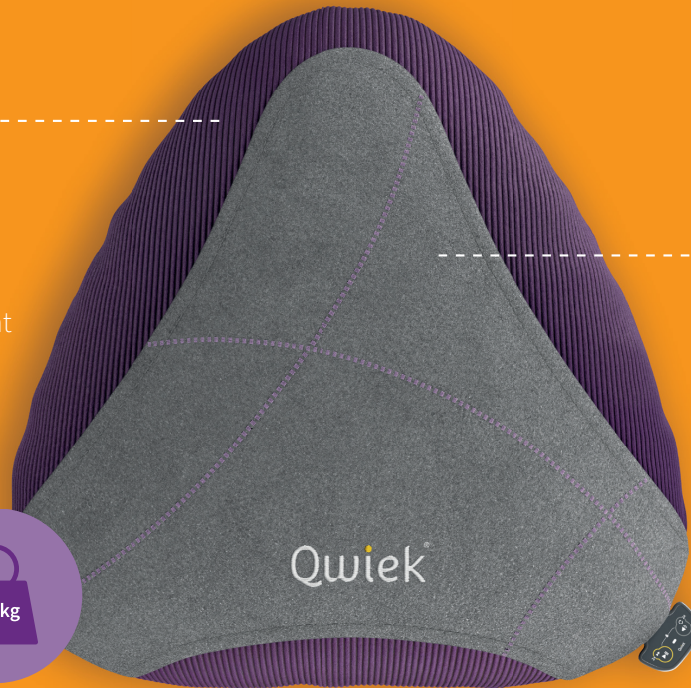

## Rhythmic vibrations

Soft, pulsing vibrations enhance body awareness.

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